# AKA

M A S K

Automatisch absorbierender Filtertyp Anti-Fog / Automatic Absorbing Filter Type Anti-Fog

Verbesserte Atemschutzmaske / Enhanced Respirator


## AKA C2 FFP2 NR

### Atemschutzmasken-Gebrauchsanweisung Half Mask User Instruction

## SCHUTZMASKE PROTECTIVE MASK FFP2 NR

EN149:2001+A1:2009

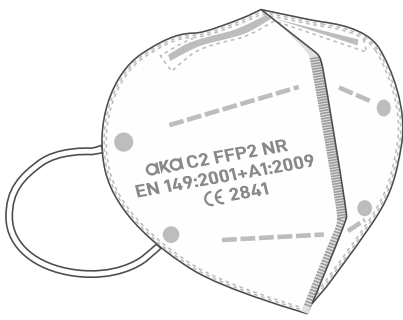MASK / MASQUE / MASKE / MASKEER  
MASCHERE / MASCARA / MASCARAR

- Hohe Filtrationseffizienz
- Niedriger Atemwiderstand
- Bequem zu tragen
- More Comfortable to Wear
- Low Respiratory Resistance
- High Filtration Efficiency

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### HOW TO WEAR YOUR MASK

|  |                                                                                                                      |   |
|--|----------------------------------------------------------------------------------------------------------------------|---|
|  | <b>Place your mask</b><br>Place the mask with the inner surface facing up. Place it on your chin and over your nose. | 1 |
|  | <b>Put on ear bands</b><br>Put the ear bands right & left over both ears at your own discretion                      | 2 |
|  | <b>Adjust the nose clip</b><br>Use both hands to carefully adjust the nose piece to your nose.                       | 3 |
|  | <b>Check fit</b><br>Make sure the mask completely covers your nose, mouth & chin & fits snugly.                      | 4 |

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### USER INSTRUCTIONS

|   |                                                                                                                                                  |  |
|---|--------------------------------------------------------------------------------------------------------------------------------------------------|--|
| 1 | <b>Do not touch</b><br>Do not touch the mask outside in the danger area. In this case, immediately put on a new mask.                            |  |
| 2 | <b>Exchange your mas</b><br>In case of feeling wetness and humidity either inside or outside of your mask, please put on a new mask immediately. |  |
| 3 | <b>Maximum wearing time</b><br>The maximum time that you can wear your mask for one time is 8 hours. Please do not exceed maximum using time.    |  |
| 4 | <b>Taking the mask off</b><br>Take off the mask only by pulling down the ear straps.                                                             |  |

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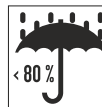
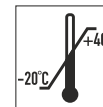
### Note:

Masks should NOT be worn by children under the age of 2 or anyone who has trouble breathing, is unconscious, incapacitated, or otherwise unable to remove the mask without assistance.

This mask must be immediately changed when it is damaged or wet.

This mask is just for single use.

Do not reuse the mask. We recommend the maximum application time of 8 hours not to be exceeded. Do not wash, disinfect, or heat the mask.

People who are allergic to polypropylene must not use this mask.

### Safety Equipment

- Control the mask before get in the using area.
- In working area there must be at least %19,5 oxygen otherwise mask not able to use.
- The masks do not protect you from gases and steam
- Change the mask if there will be any obvious and realizable extra air resisting.
- If there is any leakage redo the 1-2-3-4 process again.

Notifizierter Körper/Notified Body  
MNA Laboratuvarları San. Tic. Ltd. Şti. NB2841

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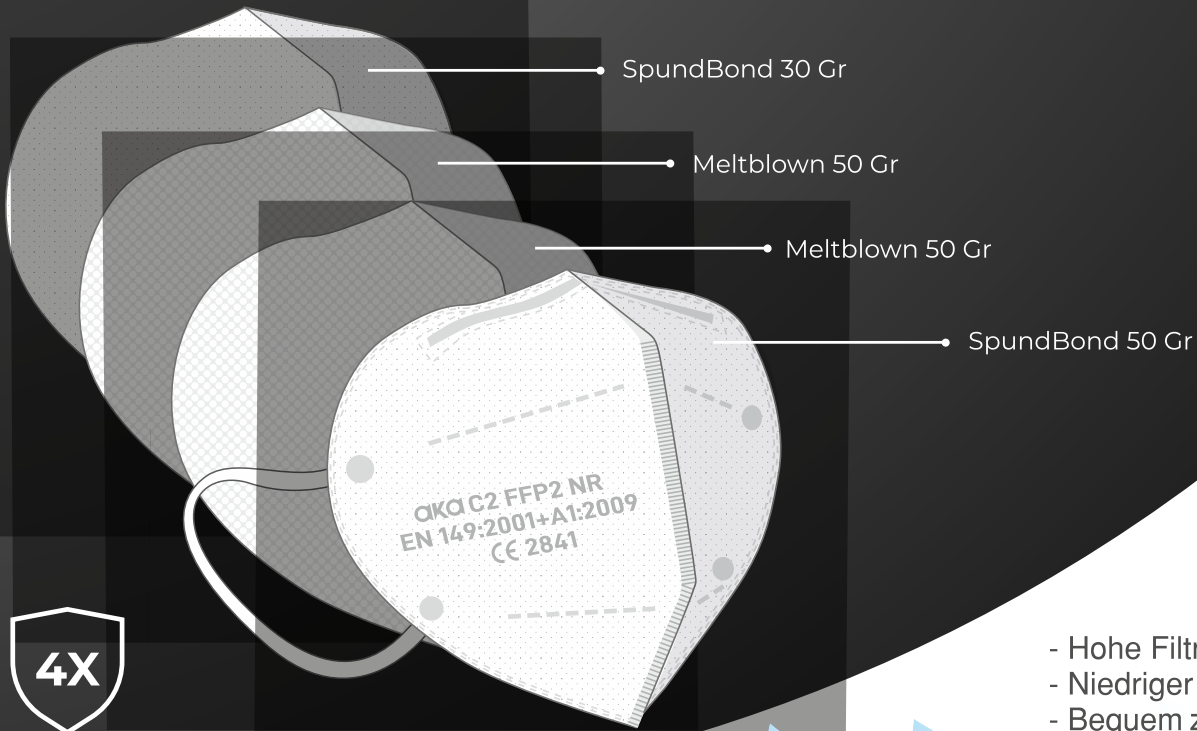
- More Comfortable to Wear
- Low Respiratory Resistance
- High Filtration Efficiency

## PROTECTIVE MASK SCHUTZMASKE FFP2 NR

EN149:2001+A1:2009

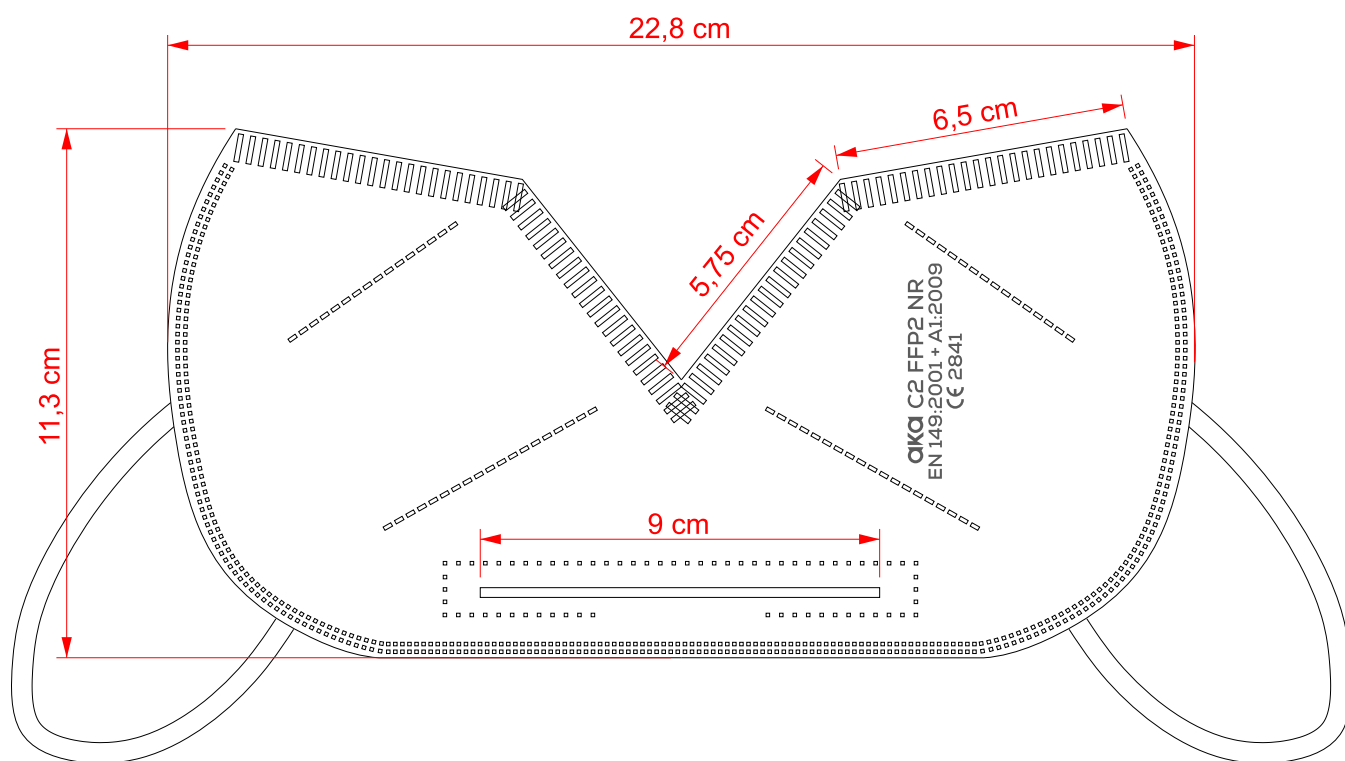
MASK / MASQUE / MASKE / MASKEER  
MASCHERE / MASCARA / MASCARAR


CE 2841 EN 149 : 2001 + A1 : 2009


# C2 FFP2 NR

# C2 FFP2 NR

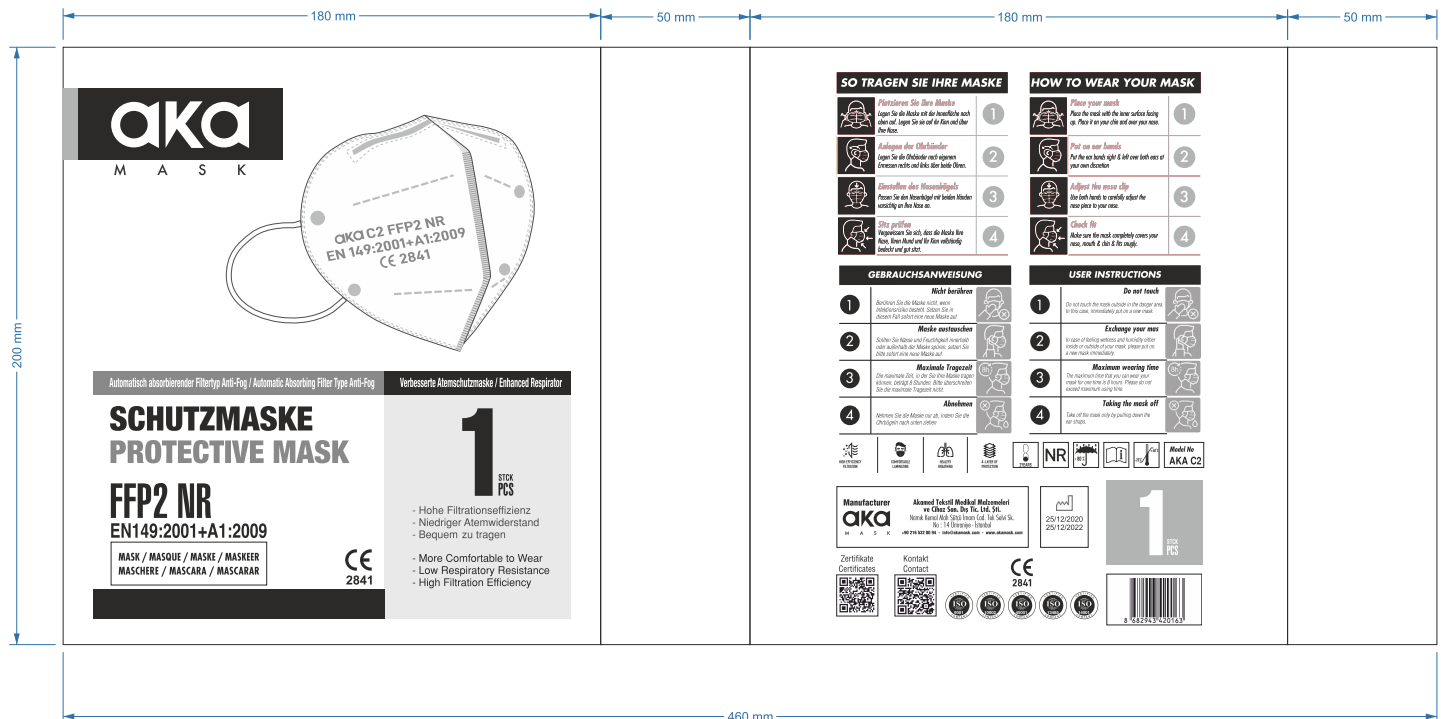**C2 FFP2 NR**
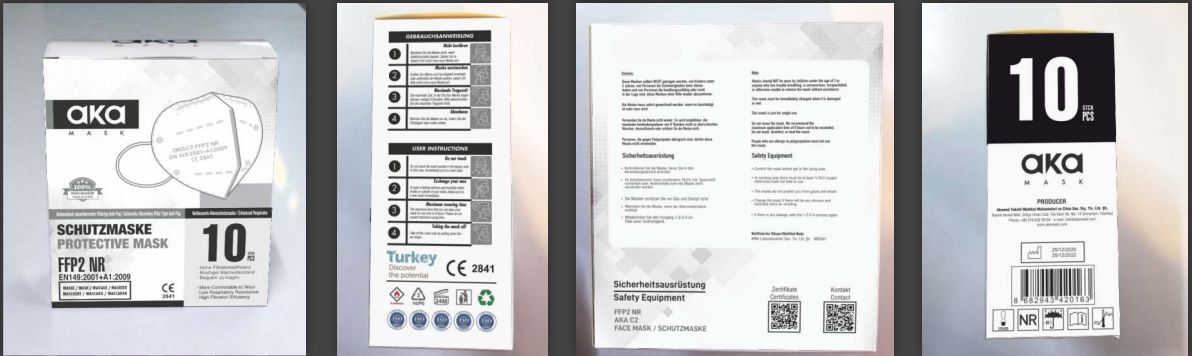
# C2 FFP2 NR

|             |           |           |           |
|-------------|-----------|-----------|-----------|
| 1910,00 mm  |           |           |           |
| 595x360x575 |           |           |           |
| 180,00 mm   |           |           |           |
| 575,00 mm   |           |           |           |
| 180,00 mm   |           |           |           |
| 595,00 mm   | 360,00 mm | 595,00 mm | 360,00 mm |
| 937,50 mm   |           |           |           |

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ve Cihaz San. Dış Tic. Ltd. Şti.
Grosshandel Hoffmann GmbH & Co.KG -